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
selvas
Healthcare


Body Composition Analysis

	Values	Body Water	Soft Lean Mass	Fat-free Mass	Weight
Body Water	49.0 (41.9 ~ 51.2)	49.0	62.2 (53.5 ~ 65.3)	66.8 (57.4 ~ 70.2)	84.6 (65.7 ~ 88.9)
Protein	13.2 (11.5 ~ 14.1)				
Minerals	4.6 (4.0 ~ 4.8)				
Body Fat	17.8 (10.8 ~ 16.2)				

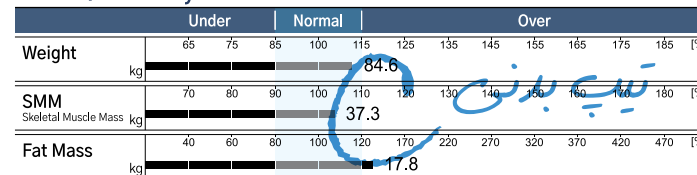
Muscle/Fat Analysis


Obesity Analysis

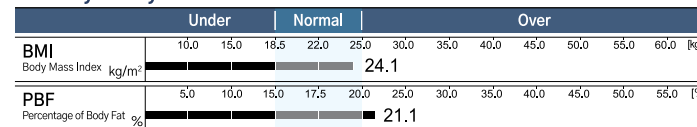Abdominal Obesity Analysis

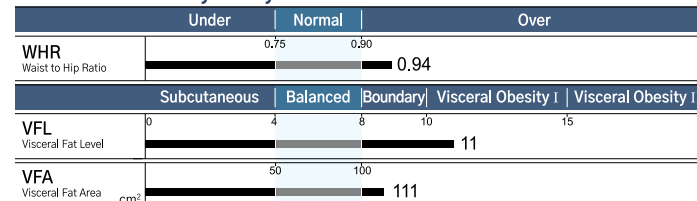Segmental Lean and Fat Analysis

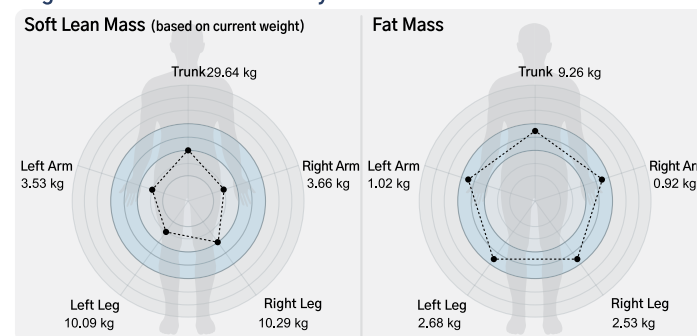Body Composition Change

Weight	84.6
SMM	37.3
PBF	21.1
Test date	2018.09.15 (14:00)

Comprehensive Evaluation

Body Type	over fat class 1
Biological Age	52 years
Basal Metabolic Rate(BMR)	1812 kcal
Total Daily Energy Expenditure	2790 kcal
Body Cell Mass	43.6 kg
Visceral Fat Mass	2.5 kg
Obesity Degree	+9.4(-10.0 ~ +10.0) %
Abdominal Circumference	89.8 (Less than 102cm) cm
Total Score	82 Points

Body Balance Assessment

Upper Body L/R	☒ Balanced ☐ imbalanced I ☐ imbalanced II
Lower Body L/R	☒ Balanced ☐ imbalanced I ☐ imbalanced II

Control Guide

Target Weight	80.9	kg
Weight Control	-3.7	kg
Muscle Control	+0.0	kg
Fat Control	-3.7	kg
ECW ratio	0.379 (Normal)	

Segmental Lean Mass (based on standard weight)


Right Arm	3.66 kg [3.27 ~ 4.00] / Normal
Left Arm	3.53 kg [3.27 ~ 4.00] / Normal
Trunk	29.64 kg [24.59 ~ 30.06] / Normal
Right Leg	10.29 kg [9.03 ~ 11.03] / Normal
Left Leg	10.09 kg [9.03 ~ 11.03] / Normal

Impedance(573)

Freq	5K	50K	250K
RA.Imp.	334	290	258
LA.Imp.	346	301	272
Trunk	27	23	19
RL.Imp.	288	250	227
LL.Imp.	295	256	232

Blood Pressure Analysis

Systolic	Lt 125 mmHg / Rt 111 mmHg
Diastolic	Lt 65 mmHg / Rt 69 mmHg
Pulse	76 bpm
Blood pressure difference between right arm and left arm	
Systolic	14mmHg, Diastolic 04mmHg

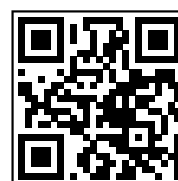For history management, please upload this results at the website using QR code scanning.

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SELVAS
Healthcare

1 Body Composition Analysis

	values	Body Water	Soft Lean Mass	Fat-Free Mass	Weight
Body Water (L)	23.1 (Normal)	23.1	29.2 (Normal)	31.2 (Normal)	40.0 (Over)
Protein (kg)	6.1 (Normal)				
Minerals (kg)	2.1 (Normal)				
Body Fat (kg)	8.8 (Over)				

2 Muscle/Fat Analysis

	Under	Normal	Over
Weight [kg]	50 70 85 100 115 140 160 180 200 220 240 260		
SMM	70 80 90 100 110 120 130 140 150 160 170 180		
Fat Mass	40 60 80 100 120 140 160 180 200 220 240 260		

3 Obesity Analysis

	Under	Normal	Over
BMI (kg/m ²)	14.5 16.5 18.5 21.7 25.0 29.4 31.6 33.8 36.0 38.2 40.5		
PBF (%)	4.0 8.5 9.0 11.5 14.0 20.4 26.8 33.2 46.1 52.5 59.0		
WHR	0.75 0.90		

4 Height/Weight Assessment

Height (cm) Weight (kg)

100 children who have a same age stand in order of height and weight. Your height ranking is 77, your weight ranking is 94. (50% is standard height and Standard weight)

[MEMO]

5 Comprehensive Evaluation

Body Type You have much body fat

Basal Metabolic Rate(BMR) 1044 kcal

Total Daily Energy Expenditure 1607 kcal

Body Cell Mass 20.5 kg

Obesity Degree +16.0 (-10.0 ~ +10.0) %

6 Body Balance Assessment

Upper Body L/R ☒ balanced ☐ imbalanced I ☐ imbalanced II

Lower Body L/R ☐ balanced ☐ imbalanced I ☒ imbalanced II

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
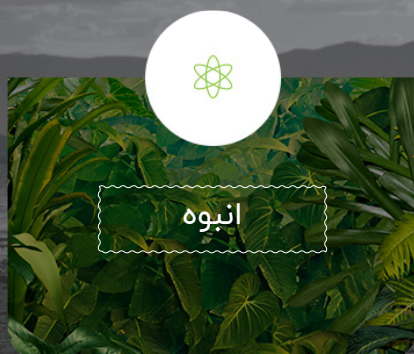
Model	ACCUNIQ BC380
Measurement Method	Tetra-polar electrode method using 8 touch electrodes
Frequency Range	5 ,50 ,250 kHz
Measurement Area	Whole body and Segmental measurement (arms, legs, and trunk)
Result Sheet Data	[Result for Body Composition Analysis] Body Composition Analysis (Weight, Lean Body Mass, Body Fat Mass, Muscle Mass, Protein Mass, Mineral Mass, Total Body Water), Skeletal Muscle / Fat Analysis (Weight, Skeletal Muscle Mass, Body Fat Mass), Obesity Analysis (Body Mass Index, Body Fat Percentage, Degree of Obesity, Abdominal Circumference), Abdominal Obesity Analysis (Waist Hip Ratio, Visceral Fat Level, Visceral Fat Area, Visceral Fat Mass), Cumulative Body Change Graph (Weight, Skeletal Muscle Mass, Body Fat Percentage), Overall Evaluation (Body Type, Body Age, Basal Metabolic Rate, Calories Needed per Day, Body Cell Mass, Total Score), Body Balance Assessment (Left and Right of the Upper Body, Left and Right of the Lower Body), Weight Control Targets (Recommended Weight, Weight Control Value, Muscle Control Value, Fat Control Value), Extracellular Water Ratio, Body Fat Mass / Muscle Mass by Body Parts (Left Arm, Right Arm, Left Leg, Right Leg, Torso), Impedance (By Body Parts and By Frequencies), Blood Pressure (When Interlocked with the Hematomanometer), QR Code [Result for Child and Youth (optional)] Body Composition Analysis (Weight, Lean Body Mass, Body Fat Mass, Muscle Mass, Protein Mass, Mineral Mass, Total Body Water), Skeletal Muscle / Fat Analysis (Weight, Skeletal Muscle Mass, Body Fat Mass), Obesity Analysis (Body Mass Index, Body Fat Percentage, Waist Hip Ratio), Child Growth Curve (Height, Weight), Nutritional Assessment (Protein, Mineral, Fat) , Overall Evaluation (Body Type, Basal Metabolic Rate, Calories Needed per Day, Body Cell Mass, Degree of Obesity), Body Balance Assessment (Left and Right of the Upper Body, Left and Right of the Lower Body), Weight Control Targets (Recommended Weight, Weight Control Value, Muscle Control Value, Fat Control Value), Body Fat Mass / Muscle Mass by Body Parts (Left Arm, Right Arm, Left Leg, Right Leg, Torso) Im pedance (By Body Parts and By Frequencies), QR Code
Measurement Current	Approx. 180 μ A $\pm$ 15
Power Consumption	60 VA

Power Supply Voltage	Input: 100-240 VAC, 50/60 Hz, 1.4-0.7 A Output: DC 12 V, 5.0 A, 60 W MAX
Display	7-inch Wide Color LCD
Input Device	Touchpad, Keypad
Transmission Device	5 USB ports, 2 RS-232C ports, Wi-Fi (basic), Bluetooth (optional)
Printing Device	USB port (printer designated by the manufacturer), thermal printer (optional)
Dimensions	Main Unit 641×436×1029 mm (W×D×H10± mm) Main Unit+Height Meter 795×436×2327 mm (W×D×H10± mm)
Weight	Approx. 18kg (main unit)
Measurement Range	100 - 950 Ω
Measurement Time	Approx. 30 sec.
Applicable Height	50 - 220 cm
Measurement Height	100 - 210 cm
Measurement Weight	10 - 250 kg
Applicable Age	1-99 years old
Operating Environment	Temperature 5 - 40°C, relative humidity 15 - 93% (no condensation)
Storage Environment	Temperature -25 - 70°C, relative humidity less than 93% (no condensation)

Optional Equipment	Ultrasonic anthropometer, fully automatic hematomanometer, ankle electrodes, result sheet for Infants, USB memory, thermal printer, and Bluetooth
Printing Logo	Printing the hospital name, address, contact information, and logo available
Touch Screen	Touch screen's sensor location adjustable
Data Storage	Up to 100,000 data units can be stored when using an ID.
Measurement Mode	Scale mode / Body composition mode
Various Result Sheets	Body composition result sheet, Result sheet for Infants (Optional)
Checking Measurement Results	LCD, Internet, ACCUNIQ app, and body composition management program (ACCUNIQ MANAGER)
USB Storage	You can save or retrieve all measurement data.
QR Code	Scan the QR code on the LCD or result sheet, transmit it to the management website, and check the results.
Remote Support	Remote technical support with PC (ACCUNIQ REMOTE SUPPORT)

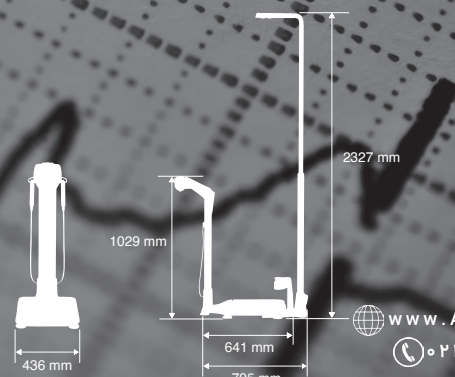※ For the purpose of improvement, the specifications and designs of this device and options may be changed without notice.
 ※ This product is a medical diagnostic device. Read "Precautions" and "Usage Methods" carefully before use.

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DEXA-ACCUNIQ	Paired T-test Analysis of Body Composition								
	Percent Body Fat (%)			Body Fat Mass(kg)			Lean Body Mass(kg)		
	Mean±SD	p-value	p-value explanation	Mean±SD	p-value	p-value explanation	Mean±SD	p-value	p-value explanation
	-0.4±0.7	0.17	DEXA PBF = ACCUNIQ PBF	-0.4±0.2	0.06	DEXA PBF = ACCUNIQ PBF	0±0.3	0.99	DEXA PBF = ACCUNIQ PBF

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